ALCANCE

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- 14101003 1.0 x 3.0mm Fixing screws
- 14101005 1.0 x 5.0mm Fixing screws
- 14101007 1.0 x 7.0mm Fixing screws
- 14101307 1.3 x 7.0mm Fixing screws
- 14101309 1.3 x 9.0mm Fixing screws
- 14101311 1.3 x 11.0mm Fixing screws
- 14101313 1.3 x 13.0mm Fixing screws

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- 14100563 1.0 mm Fixingscrew drill
- 14100569 1.3 mm Fixingscrew drill
- 14100564 Drill for boneblock
- 14100565 1.0 mm Screwdriver tip
- 14100558 1.3 mm Screwdriver tip
- 14100559 Screwdriver handpiece
- 14100566 Sterilization Box

INDICATIONS

Quick and reliable way to fix bone grafts during the surgeries.

CONTRAINDICATIONS

Contraindications include:

- 1.- Patients allergic to titanium alloy.
- 3.- Patients with insufficient mental health precluding patient cooperation
- 4.- Patients who abuse drugs and alcohol

WARNINGS

For the safe and effective use prosthetic, it is strongly suggested that specialised training be undertaken and that appropriate textbooks be studied. THESE INSTRUCTIONS ARE NOT INTENDED AS A SUBSTITUTE FOR ADEQUATE TRAINING.

Information to allergen

The materials used to manufacture the abutments are:

- Titanium Gr. 5 (Ti, Al, V, Fe, O, N, C)

PRECAUTIONS

Fixing screws are single-use. Fixing screws reuse may lead to infection and/or cross contamination. Fixing screws have to use in aseptic conditions.

CLEANING AND STERILIZATION

Before use, cleaning and sterilise the Bone Grafting Set.

The Bone Grafting Set should be cleaned, disinfected and sterilized before each use. The products are supplied non-sterile.

A proper cleaning and disinfection are prerequisites for proper sterilization.

Please do not forget that the disinfectant used in the pre-treatment used for personal protection only and does not replace the disinfection to be carried out later, once executed cleaning.

Method of sterilization

- Put the product inside retort pouches.
- Put the product inside of the autoclave. Do not sterilize different types of materials in contact.
- Sterilization must be carried out in autoclave wet heat 134 ° C (273 ° F) - 5 minutes. Make sure that the elements inside the autoclave from corrosion.
- The exposure temperature and product sterilization trays shall not exceed 137 ° C.
- Cuando el autoclave haya finalizado extraiga el producto del interior del autoclave.
- When the autoclave is complete, remove the product inside the autoclave.

SURGICAL PROCEDURES

Once the donor area is chosen, and prior to the osteotomy, a couple of holes must be done with the drill for bone blocks. Then using the drill and osteotomes the bone block can be detached. Once the bone block is released, place it against the receptor zone, use the fixing screw drill to insert the receptor bone through the holes in the donor bone block.

Sequence of use of Bone Grafting Set:

1. Marking with 1.8 mm Lindemann Drill (Ref. 13716718)

2. Cutting bone graft perimeter with 1.8 mm Lindemann Drill (Ref. 13716718)


3. Hole drilling in the graft block with the Graft Block Drill (Ref. 14100564)

4. Releasing of Graft Block with the Curved Osteotome (Ref. 14001031)

5. Receptor site drilling through the Graft Block with the Drill for 1.0 mm Screw (Ref. 14100563) or Drill for 1.3 mm (Ref. 1400569) depending on the size of the Graft Block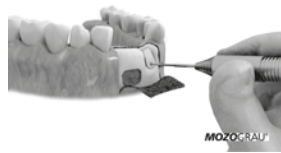

6. Screwing the corresponding screw (1.0 or 1.3 mm) different heights depending on the case

7. Graft Block properly fixed to the receptor site

8. Surgery site suture covering both the donor and the receptor site

PACKAGING AND STORAGE

TProsthetic system is supplied clean but not sterile. All information about the product is labeled. Product packaging is performed individually. Devices should be stored at room temperature. This product does not need special conditions for storage and transport, because the raw material is stable under pressure and temperature atmospheric conditions. It is recommended to store the product at temperatures entre temperatures $10^{\circ}\text{C} < T_a < 45^{\circ}\text{C}$ (bags) or $T_a < 50^{\circ}\text{C}$ (Blister).

LABELLING SYMBOLS

REFERENCE

LOT

MANUFACTURER

PAY ATTENTION TO INSTRUCTIONS FOR USE

CE MARK

ONE USE ONLY, NO REUSABLE