MOZOGRAU®	Technical File. Section 2: INFORMATION TO BE SUPPLIED BY THE MANUFACTURER	CE MARKING 
	2.2.1 INSTRUCTIONS FOR USE (ANALOGUES)	Rev.: 01
		Date: 15/10/2024
		Página 1 de 6

ESPAÑOL

ALCANCE

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ENGLISH

SCOPE

This instruction for use has its scope in TICARE analogues manufactured by Mozo Grau.

TICARE Analogues are fitted into the dental working model embedded with it or screwed to it, ensuring that the retentive elements of the analogue connection are properly aligned. Once is placed, the technician can fabricate a cement- or screw-retained restoration using the abutment. To ensure a correct transfer of the abutment position from the master cast to the patient, an individual index can be fabricated on the cast. Analogues for 3D models are inserted into a pre-designed space in a 3D printed model to provide an accurate and reliable replica of the actual implant connection. The 3D printed model analogues are especially suitable for the intraoral scan-based digital rehabilitation flow. For single crowns the index is secured with the connection of the implant/abutment and ideally supported from adjacent teeth as well, while for bridges the abutments are splinted to one another.

TICARE Implant analogue is the product specifically designed to replicate the exact position and exact external and/or internal geometry of the chosen implant/abutment for the laboratory. After an impression is taken with the transfer, the technician will choose correct analogue onto the transfer so that the model may be poured so that it exactly replicates the implant position in the mouth.

INTENDED USE

TICARE Analogue is a replica of an implant or abutment that has to be attached in a dental laboratory working model in order to duplicate the location and restorative platform orientation of the final dental implant or abutment.

INDICATIONS FOR USE

TICARE Analogues is indicated for the following medical conditions:

- Oral endosseous implantation in the upper and lower jaw to restore chewing function.
- Aesthetic dental rehabilitation.

The analogue is intended for dental laboratory use only. The analogue is compatible with dental laboratory working models. Analogues are inserted into a pre-designed space in a model to provide an accurate and reliable replica of the actual implant connection. The TICARE digital flow libraries are specially designed to leave the appropriate space for the TICARE digital analogues.

The 3D model analogues are available on the Osseous and InHex mini, standard and maxi platforms and are compatible with all the prosthetics in the TICARE catalogue.

Implant analogues:

No Digital Implant analogues are dental implant replicas that have to be attached in a dental laboratory working model in order to duplicate the location and restorative platform orientation of the final dental implant.

Digital Digital implant analogues are especially suitable for the digital rehabilitation flow based on intraoral scanning, working as non-digital implant analogues but on a digital-based model.

Abutment analogues:

No Digital Abutment analogues are dental abutment replicas that have to be attached in a laboratory working model in order to duplicate the location and restorative platform orientation of the final dental abutment.

Digital Digital abutment analogues are especially suitable for the digital rehabilitation flow based on intraoral scanning, working as non-digital abutment analogues but on a digital-based model.

MOZOGRAU®	Technical File. Section 2: INFORMATION TO BE SUPPLIED BY THE MANUFACTURER	CE MARKING 
	2.2.1 INSTRUCTIONS FOR USE (ANALOGUES)	Rev.: 01
		Date: 15/10/2024
		Página 5 de 6

Wrench analogues These abutment analogues work as non-digital abutments but include a handle for convenient clamping and retouching of the abutment or crown in the clinic and in the laboratory.

USER

TICARE Analogues should only be used by trained dental professionals

INTENDED TARGET GROUPS

TICARE Analogue is to be used in a replica of an edentulous or partially dentate adult patient’s mandible.

CONTRAINDICATIONS

TICARE analogues are for single-use only.
TICARE analogues are not sold sterile.
The 3D printed models are intended for the completion of rehabilitation anatomy in the laboratory and are not suitable for testing passive fit.
For proper fit and positioning of TICARE digital analogues use only original TICARE libraries.
The products must be used only for the function for which they were specified. It is essential to use the appropriate products for each type of implant, for this consult the product catalogue.

WARNINGS

The parts must never be changed or modified. The analogues or screws must only be used and handled by dental professionals.
The torque applied to a screw for tightening a component (e.g. abutment) placed into the analogue must not exceed the recommended torque defined by the original component manufacturer.
For the safe and effective use of the TICARE Analogues, specific training and study of appropriate textbooks is strongly recommended. THESE INSTRUCTIONS ARE NOT INTENDED AS A SUBSTITUTE FOR PROPER TRAINING. The responsibility for proper patient choice, adequate training, experience in implant placement, and provision of appropriate information for informed consent rests with the practitioner. Improper technique may result in failure and/or loss of supporting bone.

PACKAGING AND STORAGE

TICARE analogues do not need special conditions for storage, because the raw material is stable under atmospheric conditions of pressure and temperature. Transport and Storage temperature <55°C and humidity <75%. TICARE analogues are stable at atmospheric conditions of transport and storage.
The product is delivered clean, but not sterile. All information about the product is on the label. Product packaging is done individually.
Store the set at a moderate temperature in a dry, dust-free and ventilated place where corrosive vapors are not produced.

DISPOSAL

The product should be disposed of in an environmentally friendly manner in accordance with local regulations. Hazardous waste of contaminated devices or sharp objects should be disposed of in suitable containers which meet the specific technical requirements.

DETECTION AND NOTIFICATION OF INCIDENTS WITH HEALTH PRODUCTS

Important notice: Any patient or healthcare personnel who detects any serious incident related to TICARE analogues must notify the manufacturer, Mozo Grau S.A. and the competent authority of the Member State in which the user and/or patient are established.

LABELLING SYMBOLS

MOZOGRAU®	Technical File. Section 2: INFORMATION TO BE SUPPLIED BY THE MANUFACTURER	CE MARKING CE
	2.2.1 INSTRUCTIONS FOR USE (ANALOGUES)	Rev.: 01
		Date: 15/10/2024
		Página 6 de 6

SYMBOLS	DESCRIPTION
	Reference of the producto
	Batch code
	Manufacture date
	Manufacturer
	Medical device
	Non sterile
	CE Mark
	Consult electronic instructions for use
	Unique device identifier

MANUFACTURE

Mozo Grau S.A.
 C/ Santiago López González, N°7
 47197 Valladolid (Spain)
 Tel: +34 983 309 602
 e-mail: info@ticareimplants.com
www.ticareimplants.com